

# COVID-19 Screening

A volunteer will be assisting you with this process through the passenger window, please have your window down.

- A test kit will be provided
- Take your mask off of one ear
- Sanitize your hands with a wipe
- Open bag, retrieve funnel and vial
- Write your birth month and day on the vial (mm/dd), do not write on the barcode
- Remove the cap from the vial; keep your cap handy
- Attach the funnel securely to the vial
- Tilt your chin down and deposit 1 to 1.5 ml of saliva into the funnel and vial
- Remove the funnel, then screw the cap on the vial and seal tightly
- Put your mask over your mouth and nose
- Sanitize the vial and your hands using a wipe
- Place your vial in the small biohazard bag included in the kit and seal
- Place funnel and used wipes in the larger ziploc bag
- A volunteer will collect the trash and used sharpies
- Have your QR code ready
- Do not leave the area until your sample is collected.



Idaho State  
University